



## How to Connect BLE-enabled PetTest Meters to PetTest Digital Companion App:

1. **Turn on the meter** by tapping the main button (see Figure 1.1)






Figure 1.1

2. **Push and hold the main button for 3 seconds** to turn on wireless transmission. The BLE text on the meter will indicate that wireless transmission is turned on/enabled (see Figure 1.2).



Figure 1.2

3. **Open the PetTest Digital Companion App** on your smartphone, then go into “**Settings**”, then press “**Connection**”. You will find a 5 digit pairing number which should be the same as what is shown on the meter LCD screen. Next, press  button, and when  turns to , your device is connected.

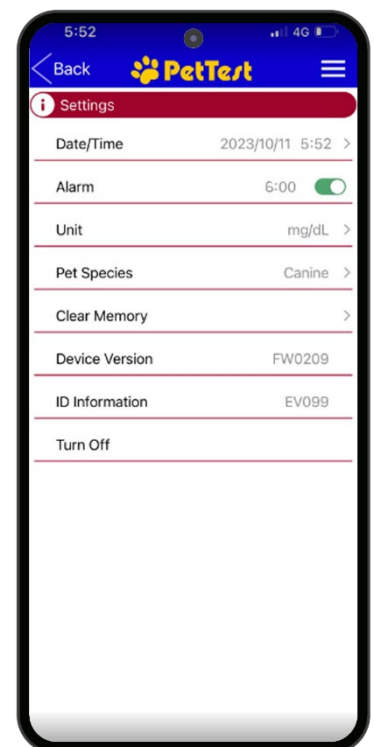


Figure 1.3

4. **Settings:** You can use the app’s Settings function to set your meter’s date/time, alarm(s), pet species, etc. You can also elect to “Clear Memory”, which will delete all meter memory/entries (see figure 1.3).

## 5. Importing meter data to app:

Data will automatically transmit from the meter to the app once connected via BLE.

"Data Import Completed" will show on screen once synced. If no data exists in the meter, a message will show that there is no data to be sent. If you wish to manually trigger another data sync, you can select "Import Data" at any time.

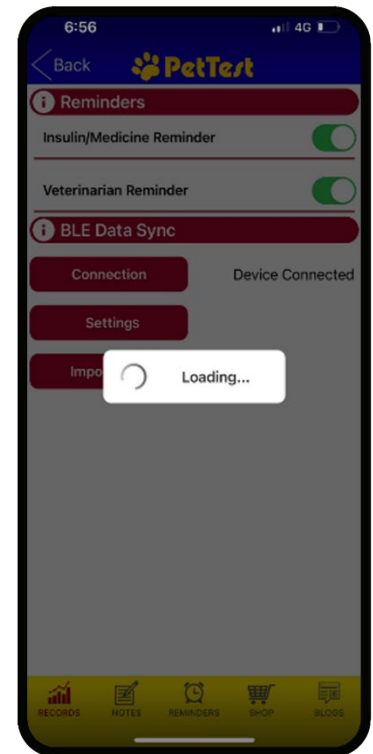


Figure 1.4

- ## 6. After testing, the meter will turn on the BLE function automatically.
- The meter screen will display the 5 digit pairing number, and then you can press the "Connection" button to connect in the app. If you don't want to connect to app at that time, simply pull the strip out of the meter. This will turn off the meter, or, if nothing is done, the meter turns off 5 minutes later automatically.



Figure 1.5